








Datasheet

Penlet



HASELMEIER
Reliability and success in therapy

Penlet at a glance

	Penlet	i-pen	i-pen ²	D-Flex	Axis-D
 Manual injection		●	●	●	●
 Auto injection	●				
 Reusable		●	●		
 Disposable	●			●	●
 Hidden needle	●				
 Multiple injections	●	●	●	●	●
 Variable dosage		●	●	●	●
 Fixed dosage	●			●	

Penlet System

The only disposable multi-injection autoinjector for 3 ml cartridges. The Haselmeier Penlet is a fully automated, multi-dose, fixed dose injection device designed for use with a standard 3 ml cartridge. The Penlet features Haselmeier's hidden needle design which hides the needle from the patient before and during injection. Pressing the clip on the pen causes the needle to automatically enter the subcutaneous tissue, followed by delivery of the fixed volume of the solution.



The Penlet incorporates the following features:

✓ Multiple fixed dose injections

✓ Easy to use


✓ Hidden needle to overcome the fear of injection

✓ No dose adjustment required


✓ Automatic needle insertion and injection for people with reduced dexterity

✓ Economical product with a cost per dose less than a prefilled syringe


Instructions for the use of Penlet

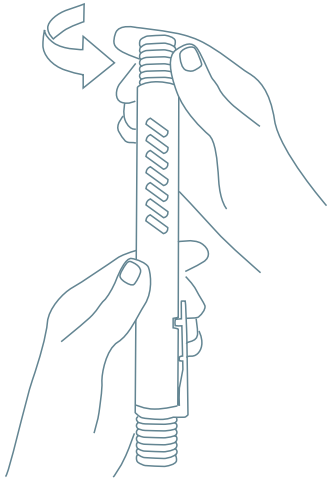


1 / Unscrew pen cap

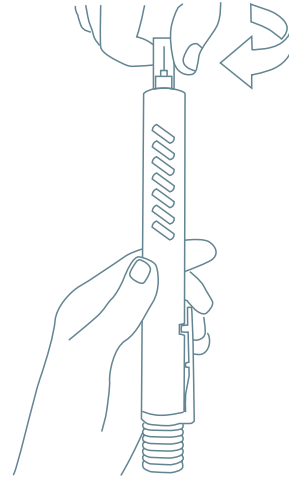



2 / Remove the paper tab and attach the needle to the pen

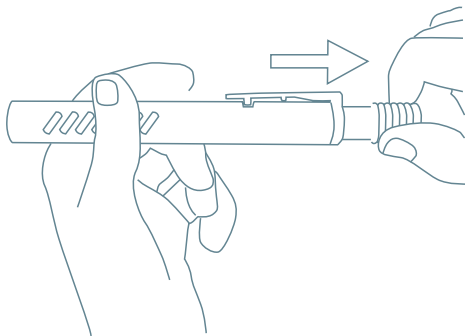


3 / Pull back the plunger until click is heard and plunger locks into position

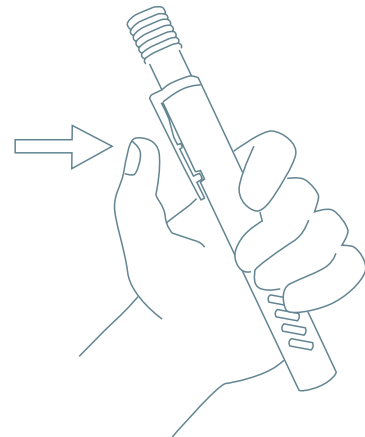

4 / Place the Penlet on injection site and depress clip

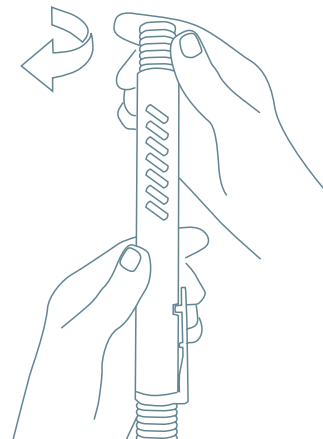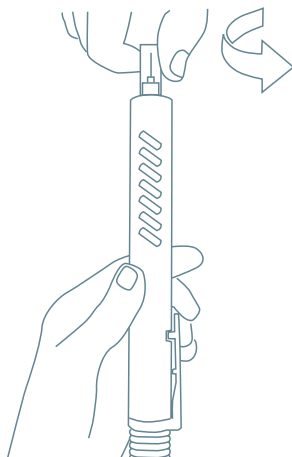


Colors

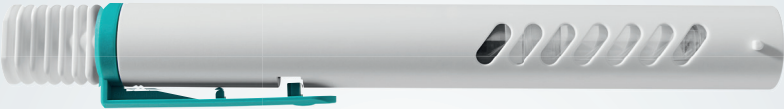

green





blue

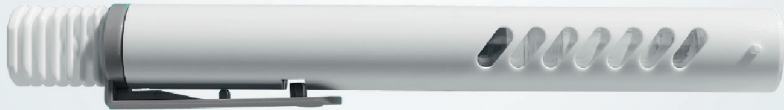

grey

Pen Dimensions

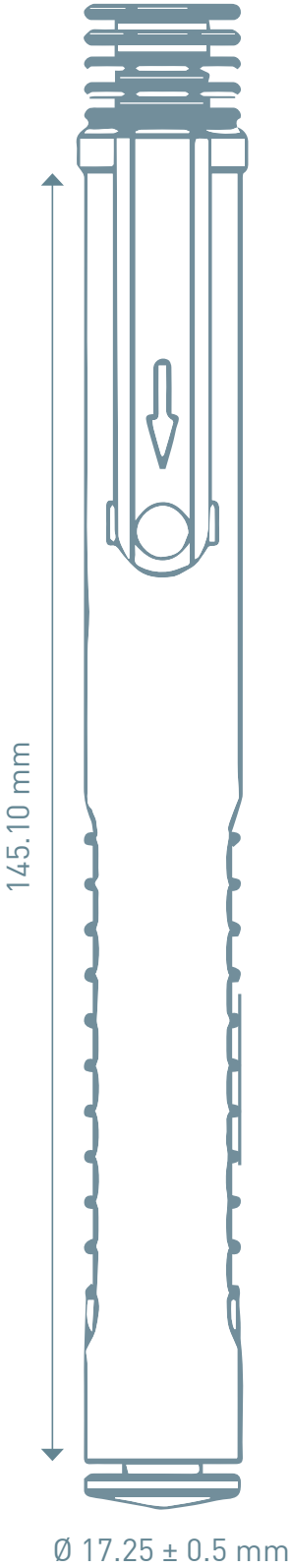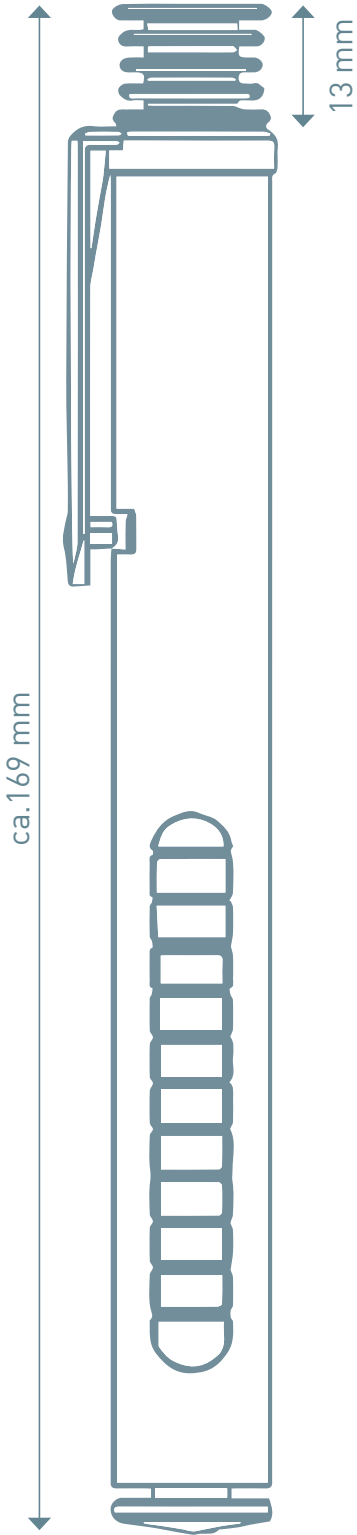

Applications

Penlet Application for various
Low-Molecular-Weight Heparins



Pen Features

✓ For injection of Enoxaparin (low-molecular-weight heparins)

✓ Deep vein thrombosis treatment

**For more information visit us at
www.haselmeier.com**

Haselmeier GmbH

Vaihinger Straße 48
DE - 70567 Stuttgart

Phone +49 711 719781-158
Email customer care@haselmeier.com

www.haselmeier.com